

# Davol Raiche Incontinence Drain advertisement.

[s.l.]: [s.n.], 1958

https://digital.library.wisc.edu/1711.dl/X77OY6DSFNP3G9D

http://rightsstatements.org/vocab/InC/1.0/

The libraries provide public access to a wide range of material, including online exhibits, digitized collections, archival finding aids, our catalog, online articles, and a growing range of materials in many media.

When possible, we provide rights information in catalog records, finding aids, and other metadata that accompanies collections or items. However, it is always the user's obligation to evaluate copyright and rights issues in light of their own use.

### **NEW! RAICHE**

## NCONTINENCE



## for female incontinence

The Raiche Drain is indwelling—with inflating balloons to assure internal and external seal of the female urethra—at the same time provide for excre-

tion of urine. Particularly adaptable for use by the non-ambulatory patient — especially helpful to spastics, paraplegies and hemiplegiacs.

#### How the Raiche Drain functions

- Drain is inserted through the urethra until the external (B) balloon is in contact with the external meatus. See diagram.
- Bladder balloon (D) inflates with approximately 20 c.c. of air or water. Catheter plug, to close off funnel, (C) is included.
- Bladder balloon is "seated" over internal urethral orifice by traction applied on the drain.
- External balloon is inflated with enough air to insure a seal over the external meatus. Funnel is sealed with plug or tied off. (A)
- Can be used for intermittent drainage or assembled to urinal reservoir for constant drainage.

\*This is a professional appliance, and should be used only under the direction and instruction of a physician.

No. 9235 — Raiche Incontinence Drain, size, 20 French — Available from your surgical supply dealer. Complete instructions packed with each Drain.



DAVOL® RUBBER COMPANY
PROVIDENCE 2, R. I.