



LIBRARIES
UNIVERSITY OF WISCONSIN - MADISON

Plastibell Disposable Circumcision Device advertisement.

[s.l.]: [s.n.], 1962

<https://digital.library.wisc.edu/1711.dl/OZTPBN4GJLQ4R82>

<http://rightsstatements.org/vocab/InC/1.0/>

The libraries provide public access to a wide range of material, including online exhibits, digitized collections, archival finding aids, our catalog, online articles, and a growing range of materials in many media.

When possible, we provide rights information in catalog records, finding aids, and other metadata that accompanies collections or items. However, it is always the user's obligation to evaluate copyright and rights issues in light of their own use.


INFANT CIRCUMCISION

CLEAN – QUICK – SAFE

PLASTIBELL®

DISPOSABLE CIRCUMCISION DEVICE


Whether you circumcise at birth or later, when you use Plastibell, the entire procedure can be done in just two or three minutes. No postoperative care or dressings are needed and the only instruments required are hemostats and scissors. The Plastibell drops off naturally after five to eight days leaving a clean, well-healed line of excision.

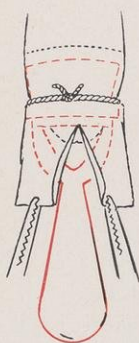

EASY TO USE . . .

Make dorsal slit, free adhesions, and place bell over glans inside foreskin. Tie suture tightly about bell, compressing foreskin into groove. Trim off foreskin. Break off bell handle.

1.3 cm. bell shown actual size. 1.1 and 1.5 cm. also available.

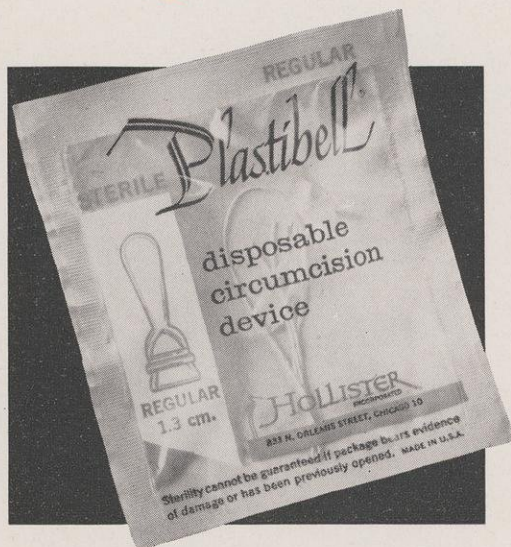

*For Free Samples
and professional
literature write to:*



HOLLISTER®
INCORPORATED

833 N. ORLEANS ST., CHICAGO 10

160 BAY ST., TORONTO 1

Plastibell and suture are sealed
in sterile packet ready to use.